

HemoCue Cleaning

Procedure

1. The cuvette holder will be cleaned daily and when obviously soiled.
2. Pull the cuvette holder out to its loading position.
3. Clean the cuvette holder with alcohol. Make sure cuvette is completely dry before replacing back in position.

The optronic unit must be cleaned on a weekly basis. To clean the optronic unit, use the HemoCue Cleaner as follows:

1. Use only the HemoCue Cleaner to clean the optronic unit. The HemoCue Cleaner should be stored unopened at room temperature until use. Check the expiration date on the cleaner.
2. Turn the Hemocue machine off and remove the cuvette holder (see operating manual if needed).
3. Remove the cleaner from the package and insert into instrument.
4. Move the cleaner over the entire optronic unit.
5. Inspect the cleaner. If it is dirty, repeat the procedure with a new cleaner.
6. Waite 15 minutes before replacing the cuvette holder.
7. Dispose of the cleaner as potentially infectious waste. Do not reuse.
8. Record optronic cleaning on cleaning log.

April 2010